

# OrthoCAD™ Digital Study Model International Shipping Instructions

Send your impressions to:  
CADENT Inc.  
640 Gotham Parkway  
Carlstadt, NJ 07072-2405, USA

## 1. Packing of Impressions

- Wrap the impressions with a wet paper towel.
- Place into a ziplock bag.
- Seal ziplock bag.
- Pack one impression per ziplock bag.
- Rubbing alcohol is not recommended for use with PVS.




## 2. Assembling Impression Boxes

- Assemble one impression box for each patient.
- Place the patients' upper and lower impressions, including bite-registration (e.g. wax-bite, bite-block, etc.), inside each impression box.


## 3. Top of the Impression Box

- Affix the office-specific label to the top of the impression box.
- Print patient name, chart number and impression date.
- Please print clearly - **Capitalize** all letters.
- On the side of the box check off the additional service(s) requested per each patient case:
  - Virtual Set-Up Model
  - Shoot the Germs Game (CD with patient game)
  - Send Plaster Casts (to request a stone cast)

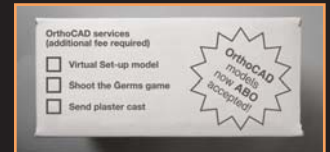

## 4. Optional: Study Model Prescription

- Fill out **optional prescription** located on the side of the impression box.
- The lab will use the information for setting the occlusion of the model.

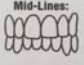Study Model Information (Optional)			
<input type="checkbox"/> CR Occlusion	<input type="checkbox"/> CO Occlusion		
Classification:	Molars: Right _____ Left _____		
	Cuspids: Right _____ Left _____		
Overjet: _____ mm	Mid-Lines: 		
Overbite: _____ mm			
Cross-bite: R _____ L _____			
Open bite: R _____ L _____			
Additional Info: _____			

## 5. International Shipment Waybill

- Fill in "impressions for digital processing" for the contents description.

**OrthoCAD™**

IT'S ABOUT TIME AND SPACE.

Cadent Inc.  
640 Gotham Parkway  
Carlstadt, NJ 07072  
Tel 800-577-8767  
Fax 866-943-7466  
support@orthocad.com  
www.orthocad.com