

OrthoCAD™ Digital Study Model Shipping Instructions

Send your impressions to:
CADENT Inc.
640 Gotham Parkway
Carlstadt, NJ 07072-2405, USA

1. Packing of Impressions

- Wrap the impressions with a wet paper towel.
- Place into a ziplock bag.
- Seal ziplock bag.
- Pack one impression per ziplock bag.
- In the Winter (November - March) - add a tablespoon of rubbing alcohol to each bag prior to shipping.

*NOTE: DO NOT add rubbing alcohol if you are using 100+ hr. impression material or PVS.

2. Assembling Impression Boxes


- Assemble one impression box for each patient.
- Place the patients' upper and lower impressions, including bite-registration (e.g. wax-bite, bite-block, etc.) inside each impression box.



3. Top of the Impression Box

- Affix the office-specific label to the top of the impression box.
- Print patient name, chart number and impression date.
- Please print clearly - **Capitalize** all letters
- On the side of the box check off the additional service(s) requested per each patient case:

- Virtual Set-Up Model
- Shoot the Germs Game (CD with patient game)
- Send Plaster Casts (to request a stone cast)

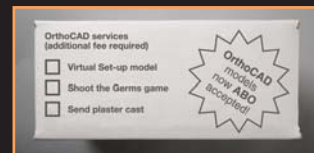

CADENT

OrthoCAD™

IT'S ABOUT TIME AND SPACE.

Cadent Inc.
640 Gotham Parkway
Carlstadt, NJ 07072
Tel 800-577-8767
Fax 866-943-7466
support@orthocad.com
www.orthocad.com

